

# Endomethasone

## Liquid

Liquid to be used with Endomethasone Powder

---

### Composition

Eugenol .....	91.0 ml
Excipient q.s.ad. ....	100.0 ml

### Indication

Liquid to be mixed with Endomethasone Powder to form a root-canal filling paste.

### Instructions for use

Prepare the root canal filling paste extemporaneously, progressively mixing 2 spoonfuls of ENDOMETHASONE Powder with 3 to 6 drops of ENDOMETHASONE Liquid, until the paste reaches the consistency required for the chosen root-canal filling technique.

Fill the root canals using a paste-carrier and insert into the paste one or several points after adapting it/them to the canal shape.  
Take X-rays to ensure that the canals have been thoroughly filled.

### Warnings and precautions for use

Eugenol inhibits the polymerization of resin materials.

Xi – Irritating (contains eugenol).  
Irritating to skin. May cause sensitization by skin contact.  
Avoid contact with skin. Wear suitable gloves.

### Presentation

10 ml bottle.

*For professional dental use only*

---

SEPTODONT  
58, rue du Pont de Créteil  
94107 Saint-Maur-des-Fossés Cedex - France  
Tel.: 33(0)1 49 76 70 00



# Эндометазон

## Жидкость

Жидкость для замешивания порошка  
Эндометазона

---

### Состав

Эвгенол .....	91,0 мл
Наполнитель q.s.ad. ....	100,0 мл

### Показания

Жидкость для смешивания с порошком Эндометазона для формирования пасты для пломбирования каналов.

### Способ применения

Приготовить пасту для пломбирования каналов непосредственно перед применением, постепенно смешивая 2 мерные ложечки порошка Эндометазона с 3–6 каплями жидкости Эндометазона до получения консистенции, необходимой для выбранной техники пломбирования.

Запломбировать корневой канал с помощью каналонаполнителя и ввести в канал один или несколько штифтов, предварительно адаптировав их к форме канала.

Рентгенологически проверить правильную obturацию каналов.

### Предостережения и меры предосторожности

Эвгенол замедляет полимеризацию композитных материалов.  
Xi – вызывает раздражение (содержит эвгенол).  
Вызывает раздражение кожных покровов. При попадании на кожу может вызвать сенсбилизацию.  
Избегать попадания препарата на кожу. Пользоваться соответствующими перчатками.

### Форма выпуска

Флакон 10 мл.

РУ № ФСЗ 2011/09765 от 24/05/2011

**Только для профессионального применения в стоматологии**

---

SEPTODONT  
58, rue du Pont de Créteil  
94107 Saint-Maur-des-Fossés Cedex - France  
Tel.: 33(0)1 49 76 70 00

